

## Multi-functional Sample Rod (SR26)

**Material:** Plastic

**Length:** 3.5 in (9.0 cm)

**Weight:** 0.35 g

### Usage

- For powder samples, use the micro scoop to collect the sample and dissolve it in water for testing. A full scoop is approximately 0.5 mg.
- For tablets, use the file to grind them into powder. Place the file with the residual powder into water to dissolve and test.
- To collect residues from surfaces, use the spatula to scrape the material, then dissolve it in water for testing.
- For crystals, use the spatula to crush them into small pieces, then collect and dissolve in water for testing.

