

3-in-1 Fentanyl/Xylazine/Nitazene test Kit

micro scoop for collecting powders



water cup



multi-functional sample collector

Spatula for scraping surfaces

← file for grinding tablets



test strip

Instruction



① fill the cup with water



tablet



powder



surface




↑ amount of sample needed

② collect the sample using the appropriate method for the sample type



③ put the sample in the water and stir with the collector

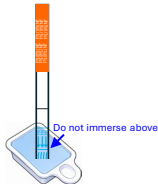

Do not immerse above

④ take a strip out of the pouch and dip into the water for 15 seconds




⑤ take the strip out and place it on a non-absorbent flat surface, wait for 5 minutes.

See reverse side for result interpretation

Scan the QR code to watch the instruction video



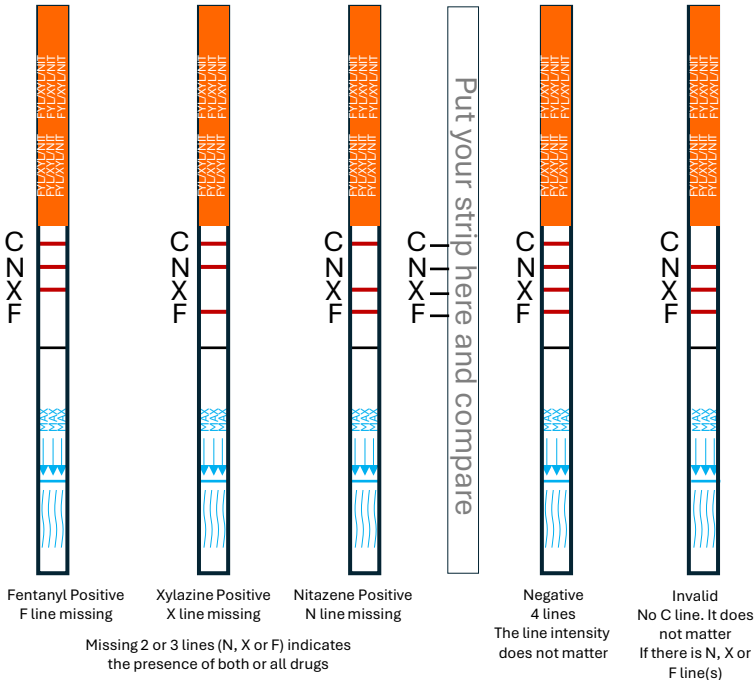

Put the strip alongside, compare and read the result. Do not read after 2 hours.